**IMAGE
IMAGEN**

GNATUS
Dental

YOU
IN A TECHNOLOGIC
WORLD.

USTED
EN UN MUNDO
DE TECNOLOGIA

X-RAYS
RAYOS X

MULTIMEDIA KIT
KIT MULTIMEDIA

NEGATOSCOPE
NEGATOSCOPIO

X-RAY/RAYOS X

SOMMO

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Extremely lightweight, with a very high degree of radiation insulation, allows a smooth, precise movement and with excellent stability.

Light Green Technology

With 95% less lead at the head of the X-Ray.

Selecting and adjusting digital and analogical time

Optional preprogrammed time available in the control of apparatus according to the need.

Pre-programmed times

20 pre-programmed times for different applications, depending on the type of patient, radiographic receiver or position of the tooth.

Time intervals

25 standardized exposure time intervals - also allows operations from centesimal takes starting on 0.06 seconds.

Reprogramming of times

The professional can enter and record new desired times.

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LIGHT GREEN
TECHNOLOGY
95% LESS LEAD
TECNOLOGIA
LIGHT GREEN
95% MENOS PLOMO

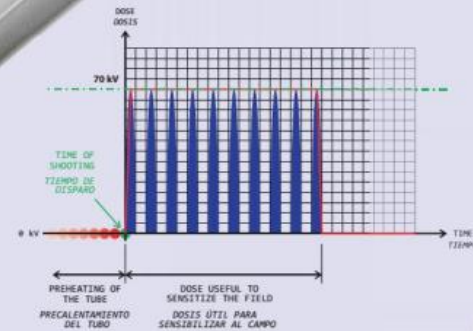
Regulator / Stabilizer

Voltage stabilizer which allows the formation of 70 KV power required for the X-Ray emission.

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Preheating of the tube

Fluctuations in the power grid are common and can impair the acquisition of radiographic images. The preheating is a system that keeps the X-Ray tube heated in the temperature optimal for shooting, providing stable and accurate results at each radiography.

Exclusive Thermostat

Controls the energy level through the filament preheating allowing a much longer lifetime of the bulb.

Dual voltage

Automatic multi-voltage
Range 100V to 260V
[recognizes the tension automatically]

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Available in different models*.
Disponibles en diferentes modelos*.

Wall Pantograph

Freedom of movement, space optimization and long range.

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Floor Pantograph

Provides overall approach of the operative field.

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*Also available with common arm (not scissors).

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Digital Control with Detachable Cable

Functions controlled by the digital control that has magnetic attachment and is connected with coiled cable of 5 meters.

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**You can also add:
RECTANGULAR
COLLIMATOR***

Easy coupling accessory.
Reduces 55% of the total area
irradiated. [Requires use of
the positioner - conventional
technique]

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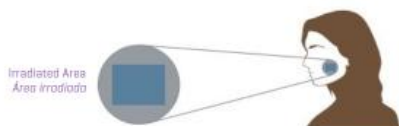
**EXTENDER
CYLINDER***

The cylinder in its elongated
shape and rectangular collimator
allow performing radiographs
with greater dimensional
accuracy.

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
Extender Cylinder
(330mm)
Cilindro Prolongador
(330mm)

Conventional Cylinder
(250mm)
Cilindro Convencional
(250mm)

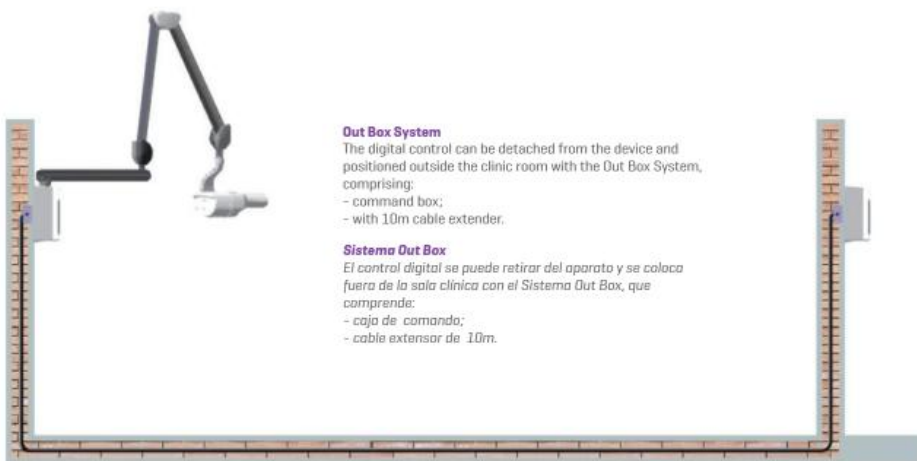*Optional item for Sommo and the Timex 70 E
*Opcionales para el Sommo y Timex 70 E

Out Box System

The digital control can be detached from the device and
positioned outside the clinic room with the Out Box System,
comprising:

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X-RAY/RAYOS X
TIMEX 70E

LIGHTNESS, STABILITY, SMOOTHNESS.
LA LIGEREZA, ESTABILIDAD, SUAVIDAD.

Exclusiva Light Green Technology
95% less lead.

Material extremely lightweight - allows smooth, precise
and excellent stability.

Highest degree of isolation from radiation.

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With 25 intervals of exposure time
set on your system, adapted to
preconceived standards.

Allows operations with centesimal
takes starting from 0.06 seconds.

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Wall
Pared

Check out the models:

Wall
Provides maximum space optimization.

Mobile Column
Base with new design, lightweight, super stable and easy to handle.

Wall Pantograph
Freedom of movement, space optimization and long range.

Mobile Column Pantograph
Allows mobility with long range.

Floor Pantograph
Provides overall approach of the operative field.

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Resolution 1366x768 pixels.

View Angle resolution - 150°.

Built-in speakers - stereo [ideal for video playback].

VGA and RCA cables included in the kit.

Remote control with functions that allows activation and distance commands.

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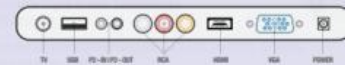
Available connections/Conexiones disponibles

*To install the Multimedia Kit, you must purchase the support kit to assemble.

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INTRAORAL CAMERA INCAM LX

Lightweight hand piece, with anatomical shape, easy to handle.

Allows you to store up to 60 images in full screen mode and 240 images in grid mode.

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NEGATSCOPE NEGATOSCOPIO

EXPANDED VISION OF YOUR DIAGNOSIS.

Extremely slim, compact and lightweight. Provides a modern and harmonious look.

Illuminated field size 20x30 cm.

Activation by side key, allowing quick and accurate access.

Polarized light, cool white, evenly distributed, providing excellent results in radiographic diagnosis.

Clear definition of contrasts.

3 clips attached for positioning and fixing of radiographs, allowing comparison diagnosis.

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Ideal for periapical radiographs, occlusal and panoramic views of up to 200x300mm.

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